

Setting up your iPulseOx monitor before your first reading



- 1 Read your owner's manual for all safety and use information.



- 2 Insert batteries into your iPulseOx monitor — be sure to use a high-quality brand like Energizer.



- 3 Insert your finger into the monitor.



- 4 Press the button to begin your reading.

Note: To change the direction of the display, simply push the button to rotate it.



- 5 Wait for the “upload success” message and you’re done! Your results are automatically sent to your provider.



SMARTMETER
Engaging Health Data

www.smartmeterrpm.com
813-773-4080
[part no. and date]